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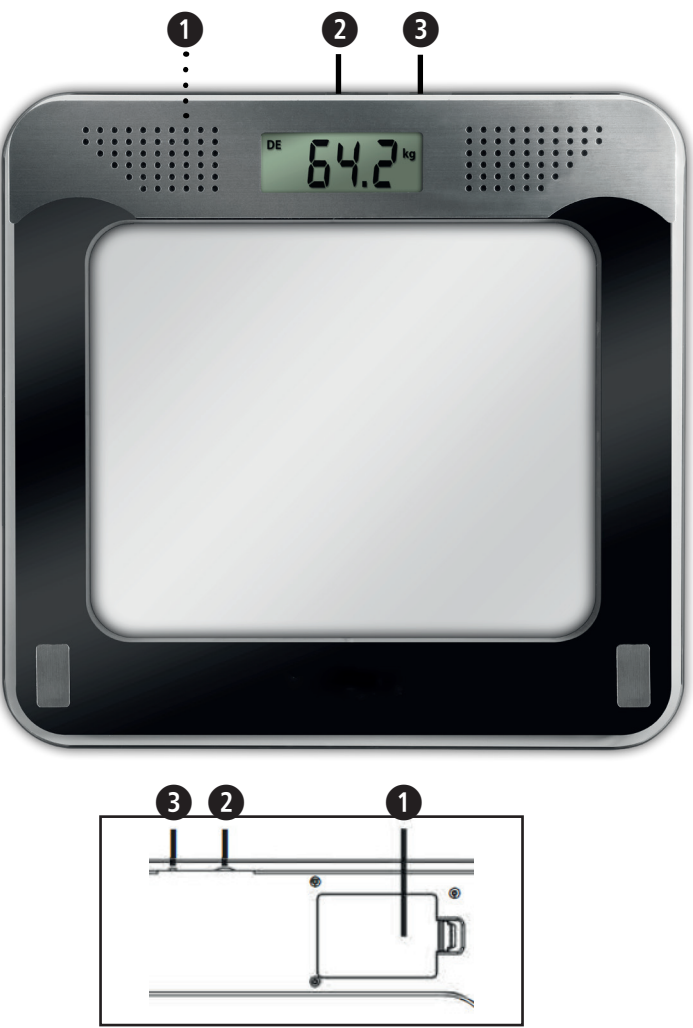
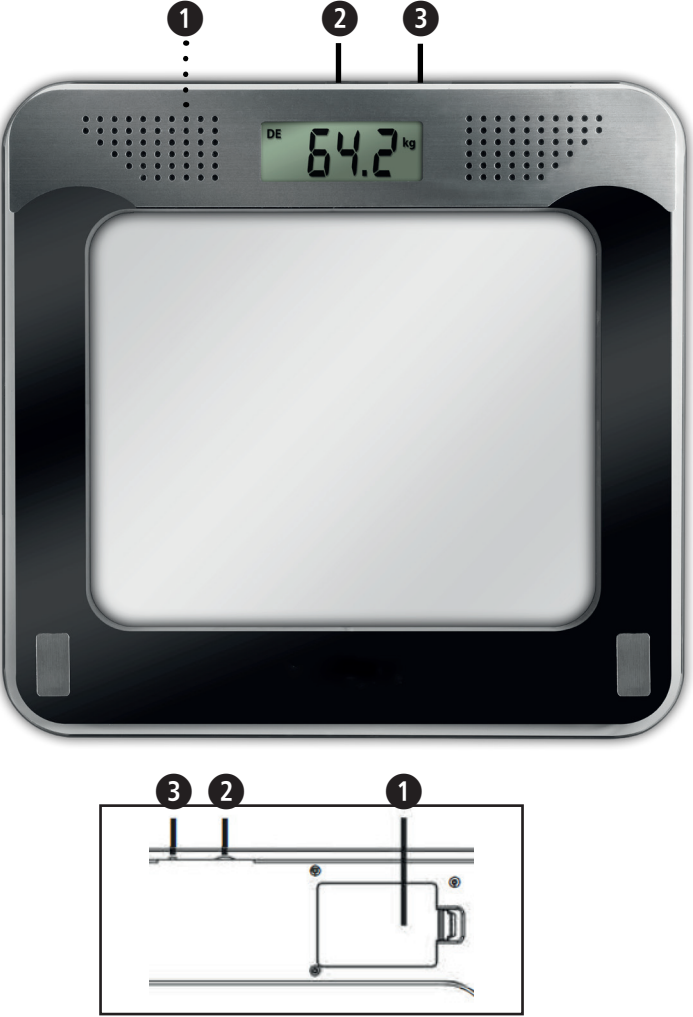
GB Personal scale with voice announcement PS 425

- 1 Battery Compartment (on underside)
- 2 Dial for setting volume
- 3 Shift key for setting language / weight unit

Explanation of symbols:

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING!
These warning notes must be observed to prevent any injury to the user.

LOT

LOT number

Manufacturer

Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.

GB Safety Information

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

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BATTERY SAFETY INSTRUCTIONS

- Do not disassemble batteries!
- Remove discharged batteries from the device immediately! Increased risk of leakage!
- Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Insert the batteries correctly, observing the polarity!
- Always keep the battery compartment well closed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!

Scope of supply

Please check first of all that the unit is complete and is not damaged in any way. If in doubt, do not use the appliance. Send it to a service point. The following parts are included:

- 1 **medisana** Digital Scales **PS 425**
- 3 Battery (Type AAA, 1.5 V)
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

Prerequisites for correct measuring results

- Always weigh yourself under the same conditions.
- Always weigh yourself at the same time of day.
- Stand up straight and stand still.
- Weigh yourself preferably in the morning, after showering or bathing, and with dry feet.

It is possible to obtain a reliable indication of body weight development from the results if measurements are carried out consistently.

Initialise the scales

Make sure the scales are on a firm, level surface. If you only want to weigh yourself but have moved the scales just before, you must first initialise the scales. To do this, press your foot on the middle of the surface of the scales. “0.0 kg” appears in the display. When the scales switch off, they are ready for the step-on function. This procedure is not necessary if you have not moved the scales.

Weighing

- Step onto the scales and stand still. The device switches on automatically and the display briefly shows „0.0 kg“.
- Your weight is measured, flashes twice and is then finally displayed.
- Step down from the scales. The scales switch off automatically after 10 seconds.

Change unit of weight

Shortly press your foot on the middle of the scale surface. „0.0“ appears in the display. You now may select the weight units (kg - lb) with the shift key for setting the weight unit **3**. Press and hold the key!

Change language

Shortly press your foot on the middle of the scale surface. „0.0“ appears in the display. With the shift key **3** for setting the language you may select the desired language for the audio output: (German [DE], English [GB], French [FR], Spanish [ES]). Press the key multiple times.

With the dial **2** you may adjust the output volume at any time. If you do not do anything else, the scale switches off after approx. 10 seconds automatically.

Insert/replace battery

Before you start up your digital scale, insert the three provided 1,5 V batteries (type AAA) into the device. To do this, open the battery compartment **1** beneath the device and insert the batteries. Make sure the polarity is correct (as shown in the battery compartment). Put the lid of the battery compartment back on and push it until it clicks into place. Change the batteries when the battery change symbol „Lo“ appears in the display or when nothing is shown in the display after the device is switched on.

Error messages

Lo = The batteries are low and need to be replaced.
Err = Check whether the batteries have been inserted correctly.

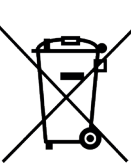
- Check whether the correct unit of weight has been selected.
- Check whether the scales are freestanding on a solid and even surface. They must not be touching a wall or any other object.
- Weigh yourself again.

O-Ld = The scale is overloaded.

Care and maintenance

Never use aggressive cleaning agents or stiff brushes. Clean the scale using a soft and lightly moistened cloth. Never use abrasive cleaning agents or alcohol. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

Technical specifications

Name and model:	medisana Personal scale with voice announcement PS 425
Voltage supply:	4.5 V (3x 1.5V, Type AAA)
Measuring range:	up to 180 kg (396 lb)
Graduation:	0.1 kg (0.2 lb)
Dimensions:	31 * 31 * 2.4 cm
Weight:	1.8 kg
Item number:	40448
EAN number:	4015588 40448 1

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and Repair Conditions

Your statutory warranty rights are not limited by our warranty set out below. In case of warranty please contact your specialist shop or the service centre directly. Should you have to return the device, please state the defect and enclose a copy of the purchase receipt or invoice.

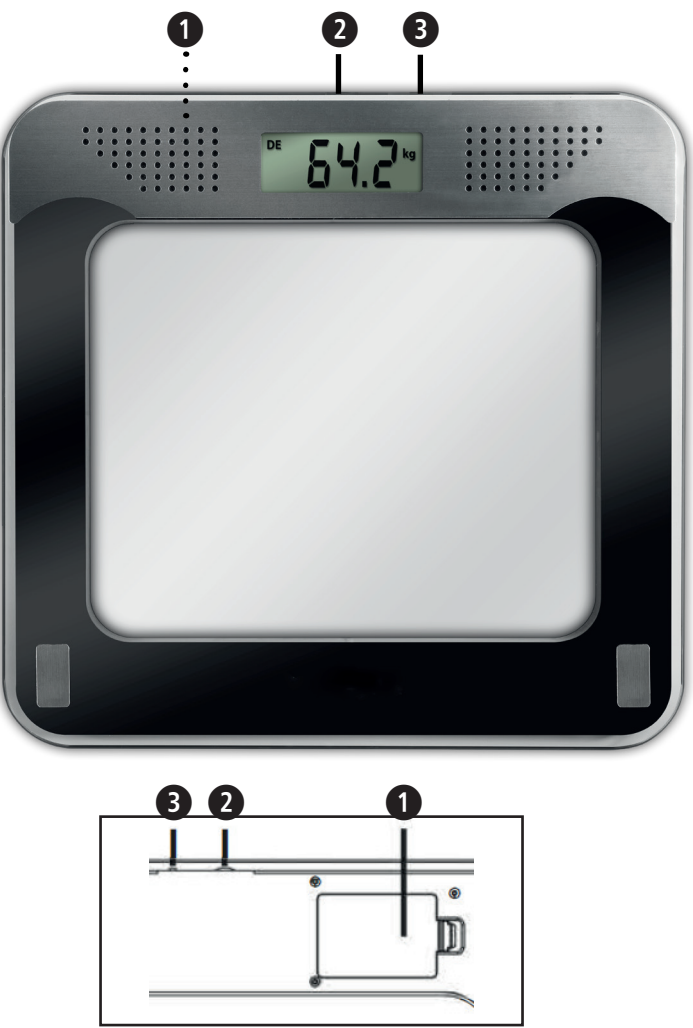
The following warranty conditions apply:

- All **medisana** products are guaranteed for 3 years from the date of purchase. In case of a warranty claim, the purchase receipt or invoice must be provided as proof of purchase date.
- Faults due to material or manufacturing defects will be rectified free of charge within the warranty period.
- A warranty service does not extend the warranty period for either the equipment or for replacement components.
- The following are excluded from the warranty:
 - any damage caused by improper use, e.g. not following the user instructions.
 - damage caused by repair or intervention by the purchaser or unauthorised third parties.
 - transport damage that has occurred on the way from the manufacturer to the consumer or has arisen during transit to the service point.
 - spare parts that are subject to normal wear and tear.
- Liability for direct or indirect consequential damages caused by the device is also excluded if the damage to the device is recognised as a warranty claim.

medisana GmbH
Carl-Schurz-Str. 2
41460 NEUSS
GERMANY

The service centre address is shown on the attached leaflet.

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