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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Device classification: type BF

LOT number

Manufacturer

Date of manufacture

Safety Information

- Use the device only according to its intended purpose as specified in the instruction manual.
- Only use genuine accessories and spare parts made by the manufacturer.
- Avoid contact with pointed or sharp objects. The warranty will be invalidated if used for purposes other than those for which it is intended.
- Do not attempt to repair the device yourself in the event of a malfunction. Repairs should only be carried out by authorised service centres.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision
- Hold the device as upright as possible during use.
- Do not operate the device in the vicinity of anaesthetic mixtures that are inflammable in air or oxygen.
- Do not operate the device in the vicinity of high-frequency electromagnetic transmitters.
- Never use the device whilst taking a bath or shower.
- When cleaning, do not hold the whole device or the nebulizer chamber under running water. Take care to ensure that no water gets inside the device.
- Do not leave the device outside or in a humid environment. The device is not protected against the ingress of liquids.
- If you are not going to use the device for a lengthy period, take out the batteries to prevent them leaking.
- Before beginning therapy with the device, consult your doctor regarding dosage, as well as the length and frequency of use.
- **The nebulizer is not suitable for using in respiration systems for anaesthesia or for artificial respiration systems.**
- Consult your family doctor before using the device if you have diabetes or other illnesses.
- Pregnant women should observe the necessary precautions and their personal circumstances; consult your doctor if necessary.
- Do not use the device if the application will come into contact with bruises, burns, inflammations, rashes, wounds or sensitive areas.
- The treatment should be pleasant. Stop using the device and consult your doctor if you feel pain or using the device is un-pleasant.

Thank you

Thank you for your confidence in us and congratulations!

By purchasing the ultrasonic nebulizer **USC** you have acquired a quality product by **MEDISANA**.

To derive the most success and to enjoy your device for a very long time, we recommend that you read the following information on the use and care of the device carefully.

2.1
Items supplied
and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your dealer or your service centre.

The following parts are included:

- 1 Nebulizer **USC** with nebulizer unit, active ingredient tank and cover
- 1 Mouthpiece
- 1 Joining piece with mask
- 1 Adults' mask
- 1 Children's mask
- 4 Batteries 1,5 V, LR6 AA
- 1 Storage bag
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

2.2
Important
information
about inhalation
therapy
Living = breathing / Breathing = living

Often, diseases of the respiratory system or the lung are lengthy, painful and life restricting. Timely, effective therapies such as inhalation therapy with aerosols can be of great help and assistance.

A gas + particle mixture is defined as an aerosol; mist is an example of a natural aerosol. Aerosol therapy delivers medicines in an "air transported" form (e.g. nebulized) which are inhaled. The great advantage of this therapy (unlike taking tablets for example) is that the medicines are administered at the precise point where they will have greatest effect (via the oro-nasal-pharyngeal cavity into the lung) without passing through other organs:

1. The effect is noticeably rapid.
2. There is no (significant) effect or side effect on other organs.
3. Small quantities of medicines are used.

2.3

Special features of the

MEDISANA USC

Ultrasound technology

An aerosol is generated in the ultrasonic nebulizer due to the vibrations of a piezoelectric crystal.

Thus, electric energy is transformed into mechanical oscillations which create droplets. The optimum diameter of the droplets is 1 - 5 µm (according to "Guidelines for the aerosol therapy of lung diseases", Lungenliga Schweiz). The droplets can only penetrate into the narrow branches of the upper and lower airways if they have a small diameter thus transporting and depositing the medicine where it is intended to be most effective.

Inhalant

**Important
information
when using
preparations/
medications**

NOTES

- Consult your doctor before using any medicinal products or medicines to ensure that they are suitable for your needs or alternatively only use the medicinal products/medicine prescribed for you by your doctor.
- The MEDISANA USN ultrasonic inhaler is suitable for use with all common aerosol products (aerosols) with the exception of
 - Products with solid constituents (suspensions)
 - Viscous solutions (high viscosity)
 - Volatile oils
- Do not use any volatile oils since this may lead to damage to the nebulizer diaphragm.
- Use only water-soluble products/medicines or those which have an alcohol base.
- Use only isotonic saline solution to dilute the inhalant; otherwise bronchial spasms may occur in certain circumstances.
- Do not use oil-based inhalants.
- After using the device for salt solution inhalation, it must be cleaned and rinsed out, according to section 4.2 (Cleaning and maintenance).

Accessories

There is no provision for combining with other devices.

Use only the original accessories with this device since only then can safe operation be guaranteed.

Use a separate accessory set for each user to avoid cross-infection or the transmission of harmful substances.

Leakage safety

When pouring the inhalant into the active ingredient tank ❷, only fill it to the maximum mark (8 ml). The recommended filling quantity is between 2 and 8 ml.

The **MEDISANA USC** is leakage proof. Do not, however, shake the device when the active ingredient tank is full; there is a chance that liquids could get into the electronics and destroy the device.

Automatic limit stop

The device is equipped with an automatic limit stop: the device switches off automatically after a short time if the inhalant is used down to a very low residual quantity and there is hardly any or no liquid in the active ingredient tank.

Do not operate the device if the active ingredient tank ❷ is empty.

The device also switches off automatically if the inclination of 45 ° is exceeded in any direction.

3 Operating

3.1 Assambly

1. Place the nebulizer unit ❸ on the attachment on the active ingredient tank ❷ and lock it by turning it clockwise until the gold contact electrodes face down.
2. Now place this combined module on the base of the device, so that the white catches on the left and right click into place.
3. Put on the cover ❶ so that it too audibly clicks into place on all four catches.
4. Put in the batteries.

3.2 Inserting / removing batteries

Open the battery compartment **6** on the underside of the unit by exerting slight pressure whilst sliding it simultaneously in the direction of the arrow. Insert the four batteries (type AA LR6) supplied. Remember to insert the batteries as indicated in the illustration on the base of the device.. Replace the lid, pressing it until it snaps into place.

If the orange battery indicator **4** lights up when the device is switched on, replace the batteries.

WARNING BATTERY SAFETY INFORMATION

- **Do not disassemble batteries!**
- **Remove discharged batteries from the device immediately!**
- **Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!**
- **If a battery has been swallowed seek medical attention immediately!**
- **Replace all of the batteries simultaneously!**
- **Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!**
- **Insert the batteries correctly, observing the polarity!**
- **Remove the batteries from the device if it is not going to be used for an extended period!**
- **Keep batteries out of children's reach!**
- **Do not attempt to recharge these batteries! There is a danger of explosion!**
- **Do not short circuit! There is a danger of explosion!**
- **Do not throw into a fire! There is a danger of explosion!**
- **Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!**

3.3 Service life

With new alkaline batteries, the device can run for approximately 100 minutes. This is sufficient for 5 - 8 inhalation sessions with salt solution (depending on the dosage and length of use).

As a rule, the inhalation phase using the **MEDISANA USC** is 8 - 10 minutes if the active ingredient tank is filled to its maximum of 8 ml due to the ultrasound nebulizer's high performance. However, each inhalation session may not last more than 15 minutes.

3.4 Area of application

Possible areas of application for inhalation therapy are acute diseases of the airways such as.:

Colds

Inflammations of the paranasal sinuses

or chronic diseases such as.:

Bronchial asthma

Bronchitis

NOTE

Before starting therapy, discuss the length, dosage and frequency of use with your doctor. The method of use should be appropriate to the aim of the therapy (e.g. lung treatment = use of mouthpiece and treatment of the nasal-pharyngeal area = use of mask). Consult your doctor about this aspect as well.

3.5 What can the device be filled with?

Use only water-soluble preparations/medicines or product that are alcohol based. All common inhalation solutions for ultrasound inhalation devices can be used. Salt solution is often recommended. Do not use oil-based inhalants. Ask your doctor which preparations are suitable for you.

More detailed information on the topics of "Aerosols", "Inhalation" or "Breathing Therapy" can be found in specialist books, or obtained from study groups and associations, or from your doctor.

3.6 Preparation for inhalation

Before operating for the first time or if the device has not been used for some time, clean and disinfect the nebulizer unit as well as the mouthpiece and mask including the connecting piece (also refer to "Cleaning and maintenance").

Remove the cover ❶ from the device. Open the active ingredient tank ❷ and fill it with inhalant (maximum capacity 8 ml = 8 ccm). Please observe the dosage and instructions for use on the information leaflet enclosed with the medicine you are using. Close the active ingredient tank lid again and place the cover ❶ on the device. Make sure the cover is correctly seated and sealed, i.e. that it is latched in place. Then plug the mouthpiece ❸ or the connecting piece ❷ and mask ❸ into the attachment port of the nebulizer unit ❸.

3.7

Inhalation using the mouthpiece

1. Sit as upright as possible.
2. Enclose the mouthpiece ⑨ completely with your lips.
3. Switch on the device by pressing the ON/OFF key ⑤. The equipment-on indicator lamp (green, ④) lights up. The aerosol warms up slightly during operation.
4. Now breathe in slowly and deeply through your mouth and out through your nose. You can improve the effectiveness of the therapy, if you hold your breath for a short time after breathing in. However, this is not necessary for general success of the therapy. It is only important that you keep quiet and relaxed while breathing in and out regularly. Do not inhale too fast. If you need a break, stop using the device for a short time and take the mouthpiece out of your mouth. Then put it into your mouth again (lips enclosing it tightly) and breathe slowly in and out again.

NOTE

Hold the device as upright as possible. A slight tilt does not affect its use (leakage proof). However, take care during inhalation not to tilt the device more than 45 ° in any direction.

5. An inhalation application should not exceed 15 minutes.
6. After terminating the inhalation, switch off the device by pressing the ON/OFF key ⑤ even if you have not used up all the inhalant. The equipment-on indicator lamp (green, ④) goes out.
7. The device switches off automatically after a short time (automatic limit stop) if the inhalant in the nebulizer is used up. For technical reasons a small amount remains in the nebulizer unit.
8. Pour out the rest of the inhalant. Do not use it again.
9. Clean the device immediately after each use. (Also refer to "Cleaning and maintenance").

Once you have used the mouthpiece for a certain time, two or three drops of inhalant solution may gather in the attachment pipe. However, this does not affect the operation and function of the device. Wipe off the drops with a soft cloth after use.

3.8 Inhalation using the mask

Alternatively, two inhalation masks **8** (for children and adults) and a connecting piece **7** are supplied with the device. They can be used for inhaling aerosol through the nose. This method is particularly suitable for children and elderly people.

1. Sit as upright as possible.
2. Plug the mask **8** with the connecting piece **7** onto the nebulizer unit **3**.
3. Hold the mask pressed lightly over your nose. Make sure that the mask seals around the nose without being too tight.
4. Switch on the device by pressing the ON/OFF key **5**. The equipment-on indicator lamp (green, **4**) lights up. The aerosol warms up slightly during operation.
5. Now breathe in and out slowly and deeply through your nose. You can improve the effectiveness of the therapy if you hold your breath for a short time after breathing in. However, this is not necessary for general success of the therapy. It is only important that you keep quiet and relaxed while breathing in and out regularly. Do not inhale too fast. If you need a break, stop using the device for a short time.

NOTE

Hold the device as upright as possible. A slight tilt does not affect its use (leakage proof). However, take care during inhalation not to tilt the device more than 45 ° in any direction.

6. An inhalation application should not exceed 15 minutes.
7. After terminating the inhalation, switch off the device by pressing the ON/OFF key **5** even if you have not used up all the inhalant. The equipment-on indicator lamp (green, **4**) goes out.
8. The device switches off automatically after a short time (automatic limit stop) if the inhalant in the nebulizer is used up. For technical reasons a small amount remains in the nebulizer unit.
9. Pour out the rest of the inhalant. Do not use it again.
10. Clean the device immediately after each use. (Also refer to "Cleaning and maintenance").

**NOTE**

A larger amount of aerosol remains on the mucosa of the nasal pharynx when using the mask than when using the mouthpiece.

4.1 Extra Tip from MEDISANA

The **MEDISANA USC** with its storage bag means you can take your ultrasound inhaler with you safely and cleanly when travelling. Due to its low noise level, you can easily use the inhaler without disturbing your neighbours. Especially when treating children, it has been noted that the low-noise function leads to a higher level of acceptance of use since it is considered less disturbing and frightening.

**NOTE**

If you have several inhaler users in one household, and thus several accessory sets, we recommend that they are clearly marked, e.g. with a permanent ink pen, to prevent any mix up.

**NOTE**

Carry out the cleaning instructions carefully directly after each use as described below.

4.2 Cleaning and maintenance

- Remove the batteries before cleaning the device.
- Take the attachments, cover, active ingredient tank and nebulizer unit off the device.
- When cleaning, do neither hold the assembled device nor the basic unit under running water.
- Clean the device's casing with a lightly moistened cloth that has been soaked in mild soapy water.
- You can clean the used attachments (mouthpiece, mask, connection piece) under the tap.
- Now disconnect the nebulizer unit from the active ingredient tank. To do this, turn the nebulizer unit anticlockwise and pull it off the tank attachment. Clean both parts in fresh water.
- Allow all parts to dry separately and uncovered on a cloth for 4-5 hours. The nebulizer unit should be agitated several times after rinsing to free the mesh from water drops.

- This cleaning routine should also be carried out if the device has not been used for a longish period of time and prior to using for the first time.
- In the case of infectious illnesses, the device must be disinfected every day or all attachments must be sterilised. You can also use alcohol or other common alcohol-based disinfectant sprays. Make sure that no disinfectant residues are left behind on the attachments to ensure safe inhalation when next used.
- You may also use boiling water or a vaporizer (see instructions of vaporizer) to disinfect. Put the detachable parts into a pan with boiling for about 15 minutes. Pay attention that the parts do not come in contact with the pan bottom, e.g. by using a sieve.
- Allow all parts to dry thoroughly for about 4 to 5 hours on a cloth. Additionally the nebulizer unit should be agitated several times to remove drops of water from the mesh.
- After using the device for salt solution inhalation, salt crystals may form as a residue on the fine gauze of the atomiser, which may damage the gauze. In order to avoid this, we recommend you to operate the device with clean water immediately after use (roughly 1 min), thus rinsing out the salt solution.
- If the nebulisation intensity decreases after alkaline/mineral fluids have been used, the device must be descaled. Mix one drop of vinegar with one teaspoon of water and place the mixture in the active agent tank. Start the device for approx. three minutes. Empty out any remaining fluid. Remove the active agent tank with nebulization unit from the device and rinse with fresh water thoroughly.
- Always make sure the nebulizer unit and active ingredient tank are properly dried out.
- If there is medicine solution or dirt on the gold contacts between the device and the nebulizer unit, clean them with a cotton bud soaked in ethyl alcohol.

4.3 Disposal

This product must not be disposed of together with domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

4.4

**Accessories for
MEDISANA USC**

*Ask at your specialist
dealer or call our hotline
directly.*

Mask set (2 masks + adapter) for USC

Article number 54101 EAN number 40 15588 54101 8

Mouthpiece for USC

Article number 54102 EAN number 40 15588 54102 5

Covering cap for USC

Article number 54103 EAN number 40 15588 54103 2

Nebulizer chamber for USC

Article number 54104 EAN number 40 15588 54104 9

Use only the original accessories with this device since only then can safe operation be guaranteed.

4.5

**Technical
specifications**

Name and model	: MEDISANA Ultrasonic Nebulizer USC
Power supply	: 6 V $\approx$, 4 x 1,5 V batteries AA LR6
Ultrasound frequency	: 100 kHz
Nebulizing output	: more than 0.25 ml per minute with salt solution
Average particle size	: 4,8 μ m
Filling volume inhalant	: max. 8 ml = 8 ccm
Residual volume in medicine tank	: ca. 0,2 ml
Dimensions	: approx. 142 x 75 mm
Weight	: approx. 300 g
Operating conditions	: 5 °C to 40 °C, less than 80 % relative humidity, non-condensing
Storage conditions	: -10 °C to +45 °C, less than 80 % relative humidity, non-condensing
Classification	: Device classified type BF $\overline{\text{A}}$ Classified according to MDD: IIa
Item number	: 54105
EAN number	: 4015588 54105 6

CE0297**Electromagnetic compatibility:**

The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. You can find details on this measurement data on the MEDISANA homepage www.medisana.com.

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair conditions

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA GmbH

Carl-Schurz-Str. 2

41460 NEUSS

GERMANY

The service centre address is shown on the attached leaflet.

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