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- ① Adjustable lumbar support  
 ② Control unit  
 ③ Connection for power supply cable  
 ④ Heat button  
 ⑤ On/off switch

#### Explanation of symbols

Do not use the device when it is folded!

Do not insert needles into the device!

Not suitable for children under 3 years!

Only use indoors!

The cover may be washed by hand according to the label!

Do not wash! (except removable cover)

Do not bleach!

Do not dry the device in a tumble drier!

Do not iron!

Do not dry clean

**IMPORTANT!**  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes provide useful additional information on the installation or operation.

Protection category II

**LOT** LOT number

Manufacturer

#### GB Safety instructions

**Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.**

##### Power supply

- Before connecting the device to your power supply, please ensure that it is switched off and the supply voltage stated on the rating plate is compatible with your mains supply.
- The device may only be operated with the supplied power supply unit.
- Keep the power supply, cable and device away from heat, hot surfaces, moisture and liquids.
- Do not touch the power supply if you are standing in water or if your hands are wet or moist.
- Only plug the power supply unit into a freely accessible socket.
- After use, always switch off the appliance and disconnect the power supply using the button on the control unit.
- To disconnect the unit from the power supply, always unplug the power supply unit from the socket or disconnect the power supply at connection ③. Do not pull on cables!
- Never carry, pull or turn the device by the cable or power supply.
- Make sure that no one can trip over the cable. Do not bend, pinch or twist the cable.

##### For special people

- Do not use the device on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- If you wear a pacemaker, please be aware that the electromagnetic fields emanating from this electric product may, under certain circumstances, interfere with the functions of your pacemaker. Therefore, ask your doctor and the manufacturer of the pacemaker for advice before using this product.
- If you have any health concerns, talk to your doctor before use.

##### Operating the device

- Only use the device for its intended use as in the instruction manual.
- If misused, all guarantee rights shall become null and void.
- The device is suitable for indoor use only.
- Before each use, carefully check the article for signs of wear and/or damage. Do not operate the appliance if you notice wear, damage or signs of improper use.
- If an error occurs, check that the power supply is properly established. Do not open the unit. There are no user-serviceable parts inside the unit.
- Use the product only in the correct location as described in this manual.
- The device is not suitable for commercial or medical use (e.g. in hospitals).
- Do not immerse the device in water or other liquids.
- If liquid does enter the device, disconnect the power supply immediately.
- Do not use the device in a wet environment (e.g. in the bathroom or similar).

- Do not keep or place any electrical device on a surface where it is in danger of falling into a bathtub or sink.

If there is no function AT ALL after switching on, please check whether the power supply is correctly established. If this is the case, the device is defective. Please do not use the device and contact the service center.

#### Cleaning and maintenance

- Before cleaning, disconnect the power supply and allow the appliance to cool down for at least 10 minutes.
- The cover is removable and hand washable (soak in mild soap and cold water for 15 minutes, then rinse in fresh, cold water).
- May not be bleached, ironed or dry cleaned!
- Do not use the article again until it is completely dry.
- Untwist the cable if it is twisted.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.

#### Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

#### Technical specifications

Name and model: **medisana Ergonomic cushion OL 350**

Power supply: 100-240V~, 50/60Hz, 12V 2A

Heat output approx.: 26 W

Automatic switch-off: 120 minutes

Temperature levels: 1: 40°C±5°C; 2: 50°C±5°C;

3: 60°C±5°C;

Operating conditions: Only use in dry rooms according to this user manual

Storage conditions: Laid out flat, clean and dry

Dimensions approx.: 36 x 44 x 29 cm

Weight approx.: 1,000 g

Art. number: 60274

EAN number: 40 15588 60274 0

**In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.**

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

- Press the On/Off switch ⑤ to turn on the power. The heat will turn on at the highest or last used level.
- By pressing the heat button ④ you can switch between heat levels 1, 2 or 3. After 120 minutes the device switches off automatically.
- To switch off the unit manually, press the On/off switch ⑤.

#### Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

- The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**medisana GmbH**  
Jagenerbergstr. 19  
41468 NEUSS  
GERMANY

The service centre address is shown on the attached leaflet.

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