DE Schulter-Rückenheizkissen HP 630

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes provide useful additional information on the installation or operation.

Protection category II

LOT

LOT number

Manufacturer

Do not use the heat pad when it is folded

Do not puncture the heat pad

Not suitable for children under 3 years

Only use the heat pad indoors

The heat pad can be cleaned in a gentle wash cycle at max. 40°C

Do not bleach

Do not dry the heat pad in a tumble drier

Do not iron the heat pad

Do not dry clean

- Carefully check the heat pad before each use, if you notice wear or damage.
- Do **not** use if you notice wear, damage or signs of improper use on the heat pad, the switch or the cables or if the device is not working.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Do not fold or crease the heat pad during use.
- Neither pierce the pad with, nor attach safety pins or other spiky or sharp objects to the pad.
- Do not use the heat pad on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- The heat pad must not be used by young children over the age of 3 years, unless the control unit has been adjusted accordingly by its parents or legal guardians or unless the child has been well instructed how to use the control unit properly.
- Never use the pad unsupervised.
- The device is only for private use and is not to be used in hospitals.
- Do not fall asleep while the heat pad is switched on.
- Move the switch to position **1** for any continuous use.
- Prolonged use of the heated cushion at a high setting could lead to skin burns.
- Do not cover the pad with any other cushions. The control switch should not be covered or placed on or below the pad while the unit is in operation.

- Do not sit on the heat pad. Instead, rest the pad on or against the part of the body to be treated.
- Never touch a device that has fallen into water. Unplug the unit from the mains outlet immediately.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or turn the heat pad by the mains lead and do not let the lead get tangled.
- Do not use the heat pad when wet and only use it in dry environments (not in the bathroom or similar environments).
- The controls and leads must not be exposed to any kind of moisture.
- If a fault occurs, do not attempt to repair the unit yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- The heat pad may only be operated with the control unit (HP 630) supplied.
- If the power supply cable is damaged, it may only be replaced by **MEDISANA**, an authorised dealer or by qualified personnel in order to avoid hazards.
- A damaged power supply cable must be replaced with a power cable of the same type.
- When storing the heat pad, allow it to cool down before laying out flat in a clean and dry place.
- Do not store the device with any objects covering or resting on top of it in order to avoid sharp crinkles.
- Should you have any concerns about health issues, consult your doctor before using the heat pad.
- Do not use the heat pad on parts of the body that are swollen, injured or inflamed.
- Should you experience persistent pains in the muscles and joints, please inform your doctor. Persistent pains can be a symptom of serious illness.
- If you wear a pacemaker, please be aware that the electromagnetic fields emanating from this electric product may, under certain circumstances, interfere with the functions of your pacemaker. Therefore, ask your doctor and the manufacturer of the pacemaker for advice before using this product.

- If you experience any pain or discomfort while using the unit, stop using it immediately.

2 Operating

Thank you very much for your confidence in us and congratulations!

With this **heat pad for shoulders and back** you have purchased a quality product by **MEDISANA**. To ensure the best results and long-term satisfaction with your **heat pad for shoulders and back HP 630**, we recommend that you read the following operating and maintenance instructions carefully.

2.1 Items supplied and packaging

Please check first of all that the unit is complete and is not damaged in any way. If in doubt, do not use the appliance. Send it to a service point.

The following parts are included:

- 1 **MEDISANA heat pad for shoulders and back HP 630** with removable switching unit
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

2.2 Product features

You can use the **MEDISANA heat pad for shoulders and back HP 630** to gently warm and relax various parts of the body. Using the heat pad stimulates blood circulation in hardened muscles and helps you to relax after a strenuous day. The heat pad features an electric temperature control that regulates the temperature according to the chosen setting.

The **MEDISANA heat pad for shoulders and back HP 630** features the new 4D-warmthflow-technology, which provides a higher temperature, a more intensive warmth effect and a quicker heating up process in comparison to conventional heat pads. The basic principle therefore is the additional heat emission by convection (air current). Because of this convection the breathable quality is improved, which helps you to feel better remarkably.

2.3 Operating

Make sure, the connection cable of the control unit is properly connected to the plug of the heat pad. Plug the unit into the mains outlet and move the slider control from position **0** to position **1**. The power LED will light up in orange (warm up phase) and after reaching the final temperature level it will light up in green.

After a few minutes you will be able to feel the pad getting noticeably warmer. If you require a higher temperature setting, move the switch to the next positions **2** or **3** or the maximum heat setting, position **4**. If the heat pad gets too hot for you, move the slider back to position **3**, **2** or **1**. After a short time, the heat pad will cool down. The heat pad has an accurate electronic control system. Depending on the room temperature, the LED may **change from orange to green colour and back within seconds**. This indicates the fast re-heating and is normal.

To switch the unit off, move the switch back to position **0**. The power LED will go out, indicating that the unit is switched off. Unplug the unit from the mains outlet. Move the switch to position **1** for any continuous use. The heat pad will switch off automatically after approx. 90 minutes of continuous use. To switch it back on again, move the slider switch to position **0** and then back to the desired heat setting. Unplug the unit from the mains outlet once you have finished using it.

The particular shape is especially designed for use on shoulders, neck and back. Using the heat pad stimulates blood circulation in hardened muscles and helps you to relax after a strenuous day.

The heat pad features an electric temperature control that regulates the temperature according to the chosen setting. The two snap buttons on the front part and the practical belt keep the **heat pad for shoulders and back HP 630** in its position.

3 Miscellaneous

3.1 Troubleshooting

This unit is equipped with the active protection system **APS TECH**. The unit detects different causes of errors and emits corresponding alarm sounds if necessary. In the case of an error, the power LED will light up in red.

3 beeps in 1-second interval and power LED lights up in red:

The connection cable of the control unit is not properly connected to the heat pad or another electrical defect is present. Please check the cable connection. Is the cable connected properly, please do not use the heat pad and contact the service center.

3 beeps in 2-second interval for one minute and power LED lights up in red:

The heat pad is broken, folded or fully covered. Please use the heat pad only according to this instruction manual and pay attention to all points in the safety information chapter.

Constant beeping and power LED lights up in red:

Important components are damaged or there is a short circuit. Please do not use the heat pad and contact the service center.

If there is no function at all after switching on the heat pad (power LED shows no light and no beeps can be heard), please check if the mains plug is correctly plugged in the mains outlet. If this is the case, the heat pad is defect. Please do not use the heat pad and contact the service center.

3.2 Cleaning and maintenance

Heat pad

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- The heat pad is equipped with a removable control unit (HP 630). Unplug the control unit from the heat pad by pulling out the connection cable from the plug on the heat pad.
- The heat pad can be cleaned dry (with a soft brush) or as described in the instructions on the product label.
- Never use strong detergents or hard brushes.
- To dry, lay the pad flat on a moisture-absorbent towel or mat.
- Only use the blanket again once it is completely dry.
- Reconnect the control unit to the heat pad in the correct manner (please see the sketch, the arrow markings are showing towards each other).
- You should untangle the cable from time to time.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.

3.3 Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

3.4 Specifications

Name and model	: MEDISANA Heat pad for shoulders and back HP 630
Power supply	: 230V~, 50 Hz
Heat output	: 100 watts
Automatic switch-off	: after approx. 90 minutes
Dimensions approx.	: 55 x 65 cm
Weight approx.	: 0.76 kg
Mains cable length approx.:	270 cm
Operating conditions	: Only use in dry rooms according to this user manual
Storage conditions	: Laid out flat, clean and dry
Art. number	: 61157
EAN number	: 40 15588 61157 5

In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.

The current version of this instruction manual can be found under
www.medisana.com

4 Warranty

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA GmbH

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41468 NEUSS

Germany

eMail: info@medisana.de

Internet: www.medisana.de

The service centre address is shown on the attached leaflet.

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