Zeichenerklärung / Explanation of symbols / Verklaring van de symbolen

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This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

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Schutzklasse II / Protection category II / Beschermsklasse II

LOT LOT-Nummer / LOT number / LOT-nummer

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Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.

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- Never put the unit down next to an electric stove or other sources of heat.
- Never leave the unit unattended if it is connected to the power supply.
- Do not allow the appliance to come into contact with pointed or sharp objects.
- Do not cover up the appliance when it is switched on. Never use it under blankets or cushions. There is a risk of fire, electric shock or injury.

maintenance and cleaning

- The unit is maintenance free. If a fault does occur, just check whether the mains power plug is properly connected.
- You may only clean the device yourself. If the device malfunctions please do not repair it yourself as this leads to the automatic expiry of all guarantee rights.
- Ask your specialist dealer and have repairs carried out by the service only.
- Do not immerse the device in water or other fluids.
- If, nonetheless, liquid should penetrate the device, unplug the mains cable immediately.

Device and controls

- Right and left massage pad
- Mode Selection switch with indicator light: 0 (OFF) - 1 (blue light: Massage) - 2 (red light: Massage & Heat)
- Carry handles
- On-board cord storage (bottom of unit)
- Two height-adjustable feet
- ON/OFF-switch

Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre.

The following parts are included:

- 1 medisana foot massager FM 883
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

What is the foot massager FM 883 used for?

Our feet are under great strain each and every day, making them especially vulnerable to tenseness, muscle pains and hardened muscles. The **medisana** foot massager FM 883 lets you give your feet a treat, helping them to relax after a long day. Each massage can be combined with heat treatment. This promotes relaxation, improves the blood circulation and has a soothing effect on the muscles.

CAUTION
Make sure that the maximum operating time of 15 minutes is not exceeded.

Adjusting the height

Be sure that the appliance is switched off and disconnected from the mains outlet before adjusting the height. You can adjust the height of the unit to your own particular requirements by turning the two adjustable feet **5** on the underside of the unit. There are 3 height levels. Each adjustable foot will „click“ in place when reaching one of the 3 levels.

Foot massage

Place the **medisana** foot massager FM 883 on the floor in front of a comfortable chair and plug the mains plug into the mains outlet.

- Power on the unit by switching on the ON/OFF button **6**. Sit down in a comfortable position.
- Press the Mode selection switch **2** to select massage mode. The indicator will light blue and the rotating massage heads are moving slowly. Carefully put your feet on the massage pads **1**.
- Press the Mode selection switch **2** again to activate the heat function (the indicator will light red and the rotating massage heads are moving more quickly). The massage heads light up in red. The heat is radiated through the massage heads.
- Press the Mode selection switch **2** the third time to stop using the unit (massage and warmth emission will stop).
- Press the Power switch **6** to switch off the appliance completely and unplug the appliance from the wall outlet after use.

The appliance is not designed to support your full body weight!

Back massage

Upper back massage: Turn the adjustable feet anticlockwise until you can feel resistance. You can now hang the unit over the back of a chair. A chair with a high back, such as an office chair, is best suited to this purpose.

Lower back massage: Place the massager vertically on the seat of a chair, such that the unit is leaning against and supported by the back of the chair.

- Plug the unit into the mains, making sure that the lead is not stretched. The control switches are operated in the same way as described in the instructions for foot massage.
- Once you have completed the massage/heat treatment, press the Power switch **6** to switch off the appliance completely and unplug the appliance from the wall outlet after use.

Duration of massage treatment

In general, you can allow approximately 10 to 15 minutes operation for each massage session, one to two times a day. Make sure the appliance is cooled down completely before using it again!

Cleaning and maintenance

- Before cleaning the appliance, make sure that the appliance is switched off and the power supply is disconnected. Allow the unit to cool down.
- Clean the massager with a slightly damp sponge only. Never use brushes, aggressive cleaning agents, petrol, thinners or alcohol. Rub the unit dry with a clean dry cloth.
- Never immerse the appliance in water to clean it and ensure water cannot get into the appliance.
- Wait until the device is completely dry before using it again.
- Undo any kinks in the cable.
- Take care to roll up the mains lead to avoid damaging it.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.

Disposal

This product should not be disposed of together with domestic waste. Please hand in all electrical or electronic devices at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority for information about disposal.

Technical specifications

Name and model:	medisana foot massager FM 883
Voltage supply:	220-240V~ 50/60Hz
Power consumption:	50W
Operating conditions:	Only use in dry rooms
Storage conditions:	In a cool, dry place
Dimensions:	approx. 40.5 x 36.5 x 12 cm
Weight:	approx. 3.3 kg
Article No.:	88396
EAN Code:	40 15588 88396 5

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

- The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
 - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

medisana GmbH
Carl-Schurz-Str. 2
41460 NEUSS
GERMANY

The service centre address is shown on the attached leaflet.

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