MEDISANA®

home of wellness

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GB

- 1** Fastening straps (rear)
- 2** 2 adjustable massage heads for neck/shoulder (Shiatsu massage)
with removable cover
- 3** 2 three-dimensional oscillating twin massage heads (Shiatsu massage),
optional suitable for spot or rolling massage, with additional heat function
- 4** Plug with cable
- 5** 2 Massage motors (Vibration massage)
- 6** Remote control pocket
- 7** Control unit
- 8** Demo button with LED (performs all massage functions)
- 9** On/Off button with LED (switches the appliance on/off)
- 10** Neck button with LED (switches neck/shoulder massage on/off)
- 11** Adjust-buttons ▼ ▲ (adjust the height of neck massage heads)
- 12** Spot massage button with LEDs (switches spot massage on/off)
▼ ▲ (adjust the height of spot massage heads)
- 13** Width button with LED
◀ ▶ (adjust the distance between the rolls)
- 14** Rolling massage button with LED (switches back massage on/off)
- 15** Shiatsu massage button with LED (switches back massage on/off)
- 16** Lower button with LED (switches lower back rolling or Shiatsu massage on/off)
- 17** Upper button with LED (switches upper back rolling or Shiatsu massage on/off)
- 18** Full button with LED (switches whole back rolling or Shiatsu massage on/off)
- 19** Heat button with LED (switches the heating on/off)
- 20** Vibration button with 3 LEDs (switches upper thigh area massage on/off;
3 intensity levels to choose from: low, medium or high)

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
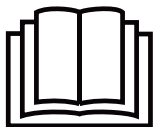
IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

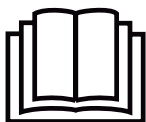Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Protection category II

LOT

Lot number

Manufacturer

about the power supply

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Only connect the mains plug to the wall socket when the device is switched off.
- Keep the power cord and the device itself away from heat, hot surfaces, damp and liquids. Avoid touching the plug or power switch with wet or even damp hands, or when you are standing in water.
- If the device falls into water, do not try to pick it up. Instead pull the plug out immediately.
- Disconnect the plug from the socket immediately after using the device.
- Never pull on the power cord to disconnect the appliance from the mains. Always pull out the mains plug!
- Do not carry, pull or rotate the device by its power cord.
- If the power supply cable is damaged, it may only be replaced by **MEDISANA**, an authorised dealer or by qualified personnel.
- Ensure that the cable will not cause tripping. The cable must not be kinked, trapped or twisted.

specific groups of people

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children must be supervised to ensure that they do not play with the device.
- Do not use this unit to supplement or replace medical treatment. Chronic complaints and symptoms may be made worse.
- You should refrain from using the Shiatsu massage seat cover **MC 820 / MC 822**, or should consult a doctor before using it, if:
 - you are pregnant,

- you have a cardiac pacemaker, artificial joints or electronic implants,
- you suffer from one or more of the following illnesses and complaints: Circulation disorders, Diabetes, Thromboses, varicose veins, open wounds, bruises, broken skin, inflammation of the veins.
- If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.
- Never leave the unit unattended if it is connected to the power supply.
- Take care when using the heater. Do not use if you have circulatory problems or on areas not sensitive to heat. **Risk of burning!**
- Take care when using the heating function. **The appliance has a hot surface.** Do not use the appliance on small children, vulnerable individuals or people who are not sensitive to heat.
- Consult your doctor if you have any questions concerning the therapeutic use of the Shiatsu massage seat cover.
- If you have unexplained pain, are receiving treatment from a doctor and/or use medical equipment, consult your doctor before using the Shiatsu massage seat cover.
- Use of the massage device is prohibited in the following cases:
 - not on children,
 - not on animals,
 - never while you sleep,
 - not in the car,
 - not while ingesting substances that reduce reaction times (e.g. painkiller medications, alcohol).

before using the unit

Before using the device for the first time, both transport locks must be removed from the back of the backrest at the top and bottom (see illustration at the front of the Instruction Manual).

- 1. Use the Allen-key supplied to turn the screws anti-clockwise and remove. (1)**

2. Dispose of the screws properly. (2)

On no account should the screws be refitted. Such action could damage the device. (2)

- Always carefully inspect the cable, the control unit and the Shiatsu massage seat cover for damage before use. Do not attempt to use a defective unit.
- Do not use the unit if there are visible signs of damage on the unit or the cable components, the unit does not work properly or the seat or the control unit have been dropped or become damp. In order to prevent hazards, always send the unit in to the service centre.
- The zip fastener on the back of the massage device must not be opened. It is only fitted for reasons relating to the manufacturing of the device.

operating the unit

- The device must only be used for its intended purpose as described in the instruction manual.
- Using the unit for any other purpose invalidates the warranty.
- Never leave the unit unattended if it is connected to the power supply.
- The maximum operating time for one session is 15 minutes. Allow it to cool off for at least 15 minutes before using it again.
- Do not allow the Shiatsu massage seat cover to come into contact with pointed or sharp objects.
- Never put the unit down next to an electric stove or other sources of heat.
- Never use the Shiatsu massage seat cover if it is folded up or crumpled.
- Never use the device under covers, such as blankets or pillows.
- Operate the device in an upright position only, never place it flat on the floor.
- Make sure that all moving parts can move freely at all times.
- Never sit or stand on the moving parts of the device with all your weight and do not place objects on the device.

The device is only intended for household use and must not be used for commercial or medical purposes. If you have health concerns, consult your doctor before using the Shiatsu massage seat cover.

The massage seat cover must only be used in enclosed spaces!

Do not use the massage seat cover in damp rooms (e.g. when bathing or showering).

maintenance and cleaning

- The unit is maintenance free.
- You may only clean the device yourself. If the device malfunctions please do not repair it yourself as this leads to the automatic expiry of all guarantee rights. Ask your specialist dealer and have repairs carried out by the service.
- Cleaning and user maintenance shall not be made by children without supervision.
- Do not use the unit if there are visible signs of damage on the unit or the cable components, the unit does not work properly or the massage seat cover or the control unit have been dropped or become damp. In order to prevent hazards, always send the unit in to the service centre.
- Do not immerse the device in water or other fluids.
- If, nonetheless, liquid should penetrate the device, unplug the mains cable immediately.

Do not wash!

Do not dry clean!

- The 100 % polyester easy clean cover of the neck massage heads **2** can be removed for washing.

You may wash the cover of the neck massage heads **2 in a normal wash at max. 30 °C.
Follow the notes on the label!**

2 Useful Information

Thank you very much

Thank you for your confidence in us and congratulations!
You have acquired a **MEDISANA** quality product with this Shiatsu massage seat cover **MC 820 / MC 822**. This unit is designed for massaging the neck, back and upper thigh areas. We recommend that you carefully read the following information on use and care of the product so that you experience its benefits and enjoy using your **MEDISANA** Shiatsu massage seat cover **MC 820 / MC 822** for many years.

2.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre.
The following parts are included:

- 1 **MEDISANA** Massage Seat with Neck Massage **MC 820 / MC 822**
- 1 Control unit
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.
If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

2.2 What is Shiatsu massage?

Shiatsu is a form of finger pressure massage and one of the most important massaging techniques along with manual lymphatic drainage and reflexology. This form of physical therapy was developed in Japan but stems from traditional Chinese massage. Gentle stroking and beneficial pressure application techniques are used to awaken life forces and encourage them to flow through the body.
The object of the massage is not just to ease tension, but also to optimise the flow of life force and harmonise body and soul.

2.3 In the **MEDISANA** Shiatsu massage seat cover **MC 820 / MC 822** you have a modern device for genuine Shiatsu style back massages. The **MEDISANA** Shiatsu massage seat cover has two neck/shoulder massage heads **2** and two three-dimensional rotating twin massage heads **3** for Shiatsu massage of the three zones "Upper Back", "Lower Back" and "Full Back". The massage heads in the back are optional suitable for spot or rolling massage. The device is equipped with an additional 3-level vibration function in the seat **5** and a heat function in the massage heads **3**. The unique combination of intensive shiatsu and vibration massage with radiated heat makes the massage seat cover an effective and relaxing treatment. Their settings can be comfortably adjusted by means of the easy-to-handle control unit **7**.

3 Operating

3.1 Usage The use of the shiatsu massage seat cover will improve your wellbeing after a stressful day. The massage effect is particularly useful for treating tense muscles and tired tissue. The unit provides pleasant relaxation for overcoming everyday stress. The massaging can improve your wellbeing, and is also useful in the sport and fitness areas.

3.2 Operation

Start-up

- Before using the device for the first time, both transport locks must be removed from the back of the backrest at the top and bottom (see illustration at the front of the Instruction Manual). Be sure to observe the safety information (see chapter **1 Safety instruction**, subheading **before using the device** and the according illustration at the front of this Instruction Manual).
- Place the massage seat cover on a high chair or armchair and attach it to this by means of the fasteners **1** on the back. Ensure that the seat cover is supported over its entire area and has a sufficiently high backrest.
- Insert plug **4** into the mains socket. Ensure that the power supply is easily accessible.
- Take a seat on the shiatsu massage seat cover and familiarise yourself with the functions.
- Sit comfortably and upright. Ensure you sit centered and such that the massage heads **3** can move to the left and right of your spinal column.

Operating the unit

The **MC 820 / MC 822** shiatsu seat cover has five functions for an effective massage: fingertip massage (Shiatsu massage), rolling massage, spot massage, vibration massage and heat. The heat function can be used together with the shiatsu or rolling massage in the back, during which the neck massage and the vibration massage in the seat can also be activated. Do not use the unit for more than 15 minutes at a time. After approx. 15 minutes, the unit will switch off automatically. Always allow the unit to cool down fully before you use it again.

Buttons on the control unit

Use the buttons on control pad ⑦ to operate the massage seat cover: Switch on the device by pressing On/Off button ⑨. The LED illuminates. The device starts with standby mode. If no other button is pressed, the device turns off after 15 minutes automatically.

Demo button ⑧

You may activate demo function with or without pressing On/Off button ⑨ before. Press the Demo button ⑧, the device starts automatically with introduction of different back setting with different functions. First massages in neck/shoulder are presented by rotating neck/shoulder massage heads ②, to be added directly by full back region massages. The rotating back massage heads ③ move from the entire bottom (start position) to the entire top. Simultaneously heat in the back and vibration in the seat are activated, while intensity of vibration is set on highest level. The LEDs show the current massage regions and functions. The device will stop once all functions worked.

Neck button ⑩

Switch on the device by pressing On/Off button ⑨. Neck button ⑩ is for the neck/shoulder reversible kneading massage. Press this button once, left massage head ② is kneading counterclockwise; right massage head ② is kneading clockwise. Press it a second time, directions change. Press button ⑩ a third time, neck massage stops.

Adjust buttons (▼ ▲) ⑪

Activate the neck massage function by pressing the neck button ⑩. Use the Adjust buttons ⑪ to position the height level of the neck massage heads ②. Press and hold the ▼ or ▲ button until the favoured position is reached.

Shiatsu Massage button ⑮

Switch on the device by pressing On/Off button ⑨. Activate the Shiatsu massage function by pressing the Shiatsu massage button ⑮. Select the region you wish to massage with the back massage button (⑯/⑰/⑱).

Lower button ⑯ (lower back)

The Shiatsu massage heads ③ travel up and down the lower half of the back in a circular motion. The reversal of the direction takes place when the respective end positions are reached.

Upper button ⑰ (upper back)

The Shiatsu massage heads ③ travel up and down the upper half of the back in a circular motion. The reversal of the direction takes place when the respective end positions are reached.

Full button ⑱ (full back)

The Shiatsu massage heads ③ move from the "Park" position at the base of the backrest, up to the shoulder area, massaging the back in a circular motion whilst doing so. Once the uppermost position has been reached, the direction of the massaging heads is reversed and the heads then travel down to the base of the backrest.

Once the heads reach this point, the direction of rotation is reversed again and then travel up the back again. Press the button 15 again to deactivate the Shiatsu massage function.

Spot message button 12 (▲ ▼)

Switch on the device by pressing On/Off button 9. Only under shiatsu state spot message for back will response. Activate the Shiatsu function state by pressing the Shiatsu massage button 15. Massage heads rotate in a spot. Pressing and holding the spot message button 12 changes position of massage heads upward or downward. Press ▲ or ▼ until the desired position is reached, loose the button 12. Press button 15 again to de-activate the spot message function and return to Shiatsu state.

Rolling message button 14

Switch on the device by pressing On/Off button 9. Activate the Rolling massage function by pressing the Rolling message button 14. Select the region you wish to massage with the back message button (16/17/18).

Lower button 16 (lower back)

The massage rolls 3 travel up and down the lower half of the back. The reversal of the direction takes place when the respective end positions are reached.

Upper button 17 (upper back)

The massage rolls 3 travel up and down the upper half of the back. The reversal of the direction takes place when the respective end positions are reached.

Full button 18 (full back)

The massage rolls 3 move from the "Park" position at the base of the backrest, up to the shoulder area, massaging the back whilst doing so. Once the uppermost position has been reached, the direction of the massaging rolls is reversed. The heads then travel down to the base of the backrest. Once the heads reach this point, the direction is reversed again and then travel up the back again.

Press the button 14 again to deactivate the rolling massage function;

Width button 13 (◀ ▶)

This button is effective only in the rolling massage state. Press the rolling massage button 14 to activate the rolling massage function. Pressing and holding the width button 13, the distance between the rolls increases or decreases until the desired width is reached and you loose the button 13.

Heat button 19

The heat function can be activated at any time with the shiatsu massage and rolling massage, during which the vibration massage can also be activated. Press the heat button 19 to activate the function; the LED on the control unit and the massage heads light up red. The heat is radiated through the massage heads 3. Press button 19 again to de-activate the heat function; the LED and the red light of the massage heads go out.

Vibration button 20

You can enhance the shiatsu massage at any time with a vibration massage for your thighs by pressing the vibration massage button 20.

The vibration massage in the upper thigh area can be operated separately. Switch on the device by pressing On/Off button 9.

Press vibration massage button 20. The LED located below left illuminates and both vibration massage motors 5 are switched on. They begin with a gentle massage. Press button 20 again and the vibration level increases (medium intensity massage). The LED located in the middle below illuminates. Press button 20 a third time and the vibration level increases (highest intensity massage). The LED located below right illuminates. After button 20 is pressed a fourth time, the vibration massage is switched off and the LED extinguishes.

Do not use the massage device for longer than 15 minutes and then allow it to cool. Using the device for longer can overstimulate the muscles and cause tension instead of relaxation.

The massage can be finished at any time by pressing the On/Off button 9.

After each use, switch the device off using the On/Off button 9 and remove the plug 4 from the mains socket. Always store control unit 7 in its bag 6.

To switch off the device

Press the On/Off button 9. The power LED begins to flash and the massage heads travel back down to the "Park" position. When this position is reached, the LED extinguishes. If you press the On/Off button 9 while one of the massage functions is switched on, the massages in neck/shoulder and seat just stop immediately, including heat function. First the massage heads in the back travel downwards to the "Park" position. Meanwhile the LED flashes and then extinguishes.

4.1 Cleaning and maintenance

- Before cleaning the device, make sure that it is turned off and the mains plug is removed from the socket. Allow the unit to cool down.
- Only clean the Shiatsu massage seat cover with a soft cloth slightly dampened with a mild detergent (e.g. soap suds). Under no circumstances should you use aggressive detergents, strong brushes, solvents, petrol, thinners or alcohol. Rub the seat dry with a clean dry cloth.
- When cleaning, never immerse the unit in water and make sure that no water gets into the unit.
- For washing the cover of the neck massage heads ② open the zip fastener all around it. Wash the cover in a normal wash at max. 30 °C and follow the notes on the label!
- Wait until the device is completely dry before using it again.
- Undo any kinks in the cable.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.

4.2 Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Consult your municipal authority or your dealer for information about disposal.

4.3 Technical Data

Name and model	: MEDISANA Massage Seat with Neck Massage MC 820 / MC 822
Voltage supply	: 220 - 240V~ 50 - 60 Hz
Power consumption	: approx. 40 W
Operating time	: max. 15 minutes
Operating conditions	: only use in dry rooms
Storage conditions	: in a cool, clean and dry place
Dimensions (L x W x H)	: approx. 118 x 40 x 9 cm
Weight	: approx. 6,5 kg
Power cable length	: approx. 1,8 m
Article No.	: 88922 (MC 822) / 88938 (MC 820)
EAN Code	: 40 15588 88922 6 / 40 15588 88938 7

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The current version of this instruction manual can be found under
www.medisana.com

Warranty and repair terms Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.
The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA GmbH

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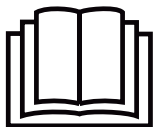
41468 NEUSS

Germany

eMail: info@medisana.de

Internet: www.medisana.com

The service centre address is shown on the attached leaflet.

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